

Poliresin

Instructions for use

The same polishing technique is used with Poliresin as with pumice powder.

Water is poured into the polishing trough and Poliresin added until a slurry is formed. It should be possible to take a handful of the mixture without it running through the fingers to ensure adequate adhesion to the denture to be polished.

Ensure that the acrylic surfaces to be polished are always kept coated with moist Poliresin to avoid overheating.

Standard brushes can be used for prepolishing. Cloth mops and goat's hair brushes are generally adequate for reducing uneven areas and also producing an extremely smooth surface.

High-lustre polishing can then be completed in the usual way.